



Medtronic

LIFEPAK[®] 20

Defibrillator/Monitor

Simple yet sophisticated for responders of many skill levels

Highly intuitive for quick, effective AED use

Easily converts to manual mode

ADAPTIV™ biphasic technology adjusts shock duration and voltage

Provides optional, escalating shocks as needed

Compact and lightweight for easy carrying and versatile use

Superb color graphics allow easy viewing from many angles

Flexible therapy options (See Options below)

Easy to maintain and service

OPTIONS

- Masimo SET[®] Pulse Oximetry
- Noninvasive pacing
- QUIK-COMBO™ electrodes or hard paddles
- Docking station



The world leader in defibrillation technology brings you the LIFEPAK 20 defibrillator/monitor, created especially for hospitals and clinics. Designed with extensive input from clinicians around the world, the 20 is simple yet sophisticated, providing a flexible and effective tool to meet diverse defibrillation needs across settings.

The 20 is highly intuitive, making it easy for infrequent AED-trained responders to quickly understand and use. Our proven Shock Advisory System™ guides the user with voice and visual prompts through each step and simple 1-2-3 operation. With an easy push of a latch, the 20 automatically converts to a manual defibrillator, including waveform displays. In manual mode, all of the advanced tools for CPR become available. The 20 offers

noninvasive pacing, Masimo SET pulse oximetry (SpO₂), ECG monitoring (3- or 5-lead), and synchronized cardioversion.

Designed for indoor use, the 20 is compact, lightweight, and easy to rush to the scene or use during transport. A convenient, optional docking station enables the device to be firmly attached to the crash cart for safe and rapid transport, or easily released—whatever the situation requires. The docking station swivels, for multiple viewing angles. Superb color graphics, easily viewed from many angles, enable clinicians to clearly see color matched waveforms and values for efficient and informed patient care decisions.

The 20 is equipped with ADAPTIV biphasic technology, which adjusts shock waveform



GENERAL

The LIFEPAK 20 defibrillator/monitor has seven main operating modes:

Manual Mode: Provides a normal operating capability for ALS users. Allows access to manual mode energy selections up to 360J, synchronous cardioversion and pacing. ECG waveform is displayed.

AED Mode: Provides a normal operating capability for BLS users. All user features are available except manual defibrillation, synchronous cardioversion, pacing, and access to archived patient records. Provides shock energy defaults up to 360J. User selectable option to display ECG waveforms and/or visual AED prompts.

Setup Mode: Allows the operator to configure the device settings.

Service Mode: Allows the operator to execute diagnostic tests and calibrations, to display device module software and hardware versions, and to display and print the diagnostic code log.

Inservice Mode: Simulated waveforms are available for demonstration purposes. The waveforms consist of short segments of realistic data, which are repeated to form a continuous waveform.

Archive Mode: Provides operator the opportunity to access records of previous patients for review, transmission, printing, editing or deletion.

Auto Test Mode: Performs daily self tests.

POWER

The device is an AC line operated device with an internal battery as backup.

AC Powered: 90–132 VAC 50/60Hz, 198–264 VAC 50/60 Hz, total power draw less than 75 Watts.

Internal Battery Backup: NiMh. Batteries charge while device operates from AC Power.

Operating Time: A new fully charged internal backup battery will provide the following prior to shutdown:

	TOTAL	AFTER LOW BATTERY
Monitoring (minutes):	120	5
Monitoring in device without Pulse Oximeter (minutes):	135	5
Defibrillation (360J discharges):	90	3
Monitoring plus pacing (Minutes at 100 ma, 60 ppm):	70	2

Typical Battery Charge Time: <2 hours when device is powered off and AC power is applied

Low Battery Indication and Message: When the device is unplugged from AC power, it switches to battery. When battery gets low, the battery detection icon is indicated with a low battery message in the status area, and a warning tone occurs.

Warmstart: With inadvertent loss of power (<30 seconds) device retains settings.

Service Indicator: When error detected

PHYSICAL CHARACTERISTICS

Weight:

- Fully featured defibrillator/monitor (pacing and SpO₂) 5.58 kg (12.3 lbs)
- QUIK-COMBO cable: .20 kg (.43 lbs)
- Standard (hard) paddles: .88 kg (1.95 lbs)

Height: 21.3 cm (8.4 in)

Width: 26.2 cm (10.3 in)

Depth: 26.2 cm (10.3 in)

DISPLAY

Size (active viewing area): 115.18 mm (4.53 in) wide x 86.38 mm (3.4 in) high

Resolution: 320 x 240 dot color active LCD.

Displays a minimum of 4 seconds of ECG and alphanumeric for values, device instructions or prompts.

Option to display one additional waveform.

Waveform display sweep speed: 25 mm/sec for ECG

DATA MANAGEMENT

The device can easily print a CODE SUMMARY™ report, including an introduction with patient information and critical event record. The summary report also includes event and vital signs log, and waveforms associated with certain events. The device can print archived patient records and has two data communication ports— infrared (IrDA) and a direct serial port, which supports a serial data cable.

COMMUNICATIONS

The device is capable of transferring data records by serial connection:

- EIA/TIA-232E compatible at 9600, 19200, 38400, 57600 and 115200 bps

MONITOR

ECG

ECG can be monitored through 3-wire or 5-wire ECG cables.

Standard paddles or therapy electrodes (QUIK-COMBO pacing/defibrillation/ECG electrodes or FAST-PATCH® disposable defibrillation/ECG electrodes) are used for paddles lead monitoring.

Compatible with LIFEPAK 12 ECG and therapy cables.

Lead Selection:

- Leads I, II and III, (3-wire ECG cable)
- Leads I, II, III, AVR, AVL, and AVF, V (c) acquired simultaneously, (5-wire ECG cable)

ECG size: 4, 3, 2.5, 2, 1.5, 1, 0.5, 0.25 cm/mV

Heart Rate Display: 20–350 BPM digital display

Out of Range Indication: Display symbol “...”

Heart symbol flash for each QRS detection

Continuous Patient Surveillance System (CPSS):

In AED mode, while Shock Advisory System is not active, CPSS monitors the patient, via QUIK-COMBO paddles or lead II ECG, for potentially shockable rhythms.

Voice Prompts: Used for selected warnings and alarms (Configurable On/Off)

Analog ECG Output: 1V/mV X 1.0 gain < 30 ms delay

Common Mode Rejection: 90db at 50/60 Hz

SpO₂

Masimo SET® Sensors

Saturation Range: 1 to 100%

Saturation Accuracy: (70–100%) (0–69% unspecified)

Adults/Pediatrics:

- +/- 2 digits (during no motion conditions)
- +/- 3 digits (during motion conditions)

Neonates:

- +/- 3 digits (during no motion conditions)
- +/- 3 digits (during motion conditions)

Dynamic signal strength bar graph

Pulse tone at the onset of the pleth waveform

SpO₂ Update Averaging Rate: User selectable 4, 8, 12 or 16 seconds

SpO₂ Measurement: Functional SpO₂ values are displayed and stored

Pulse Rate Range: 25 to 240 pulses per minute

Pulse Rate Accuracy: (Adults/Pediatrics/Neonates)

- +/- 3 digits (during no motion conditions)
- +/- 5 digits (during motion conditions)

SpO₂ waveform with autogain control

ALARMS

Quick Set: Activates alarms for all parameters

VF/VT Alarm: Activates continuous CPSS monitoring in Manual Mode

PRINTER

Prints continuous strips of the displayed patient information

Paper size: 50 mm (2.0 in)

Print speed: Continuous ECG 25 mm/sec +/- 5% (measured in accordance with AAMI EC-11, 4.2.5.2)

Delay: 8 seconds

Autoprint: Waveform events print automatically (user configurable)

Print Speed for CODE SUMMARY Reports: 25 mm/sec

FREQUENCY RESPONSE

Diagnostic: 0.05 to 150 Hz or 0.05 to 40 Hz (user configurable)

Monitor: 0.67 to 40 Hz or 1 to 30 Hz (user configurable)

Paddles: 2.5 to 30 Hz

Analog ECG Output: 0.67 to 32 Hz (except 2.5 to 30 Hz for paddles ECG)

DEFIBRILLATOR

Waveform: Biphasic Truncated Exponential.

The following specifications apply from 25 to 200 ohms, unless otherwise specified.

Energy Accuracy: ±1 joule or 10% of setting, whichever is greater, into 50 ohms ±2 joule or 15% of setting, whichever is greater, into any impedance from 25–100 ohms.

Voltage Compensation: Active when disposable therapy electrodes are attached. Energy output within ± 5% or ± 1 joule, whichever is greater, of 50 ohm value, limited to the available energy which results in the delivery of 360 joules into 50 ohms.

Patient Impedance	Phase 1 Duration (ms)		Phase 2 Duration (ms)	
	Min.	Max.	Min.	Max.
25	5.1	6.0	3.4	4.0
50	6.8	7.9	4.5	5.3
100	8.7	10.6	5.8	7.1
125	9.5	11.2	6.3	7.4

MANUAL

Energy Select: 2, 3, 4, 5, 6, 7, 8, 9, 10, 15, 20, 30, 50, 70, 100, 125, 150, 175, 200, 225, 250, 275, 300, 325, and 360 joules and user configurable sequence of 100–200, 100–300, 100–360.

Charge time:

- Charge time to 200J <4 seconds with fully charged battery
- Charge time to 360J <7 seconds with fully charged battery
- Charge time to 360J <10 seconds while not in low battery operations

Synchronous Cardioversion:

- Energy transfer begins within 60 ms of the QRS peak
- Energy transfer begins within 25 ms of the External Sync Pulse
- External Sync Pulse; 0–5V (TTL Level) Pulse, active High, > 5 ms in duration, no closer than 200 ms apart and no further than 1 second apart

AED

Shock Advisory System is an ECG analysis system that advises the operator if the algorithm detects a shockable or nonshockable ECG rhythm. SAS acquires ECG via therapy electrodes only.

Shock Ready Time: Using a fully charged battery at normal room temperature, the device is ready to shock within 16 seconds of power on, if initial rhythm finding is “Shock Advised.”

The AED mode of the LIFEPAK 20 defibrillator/monitor is not intended for use on children less than 8 years of age.

Paddle Options

- QUIK-COMBO pacing/defibrillation/ECG electrodes (standard)
- Standard adult paddles with embedded pediatric paddles (optional)
- Internal handles with discharge control (optional)
- External sterilizable paddles (optional)
- FAST-PATCH disposable defibrillation/ECG electrodes (optional)

Cable length: 2.44 meter (8-foot) long QUIK-COMBO cable (not including electrode assembly)

PACER

Pacing Mode: Demand or nondemand
Rate and current defaults (user configurable)

Pacing Rate: 40 to 170 ppm

Rate Accuracy: +/- 1.5% over entire range

Output Waveform: Monophasic, amplitude stable to +/- 5% relative to leading edge for currents greater than or equal to 40 mA, Duration 20 +/- 1 ms, Rise/Fall times <= 1 ms [10–90% levels]

Output Current: 0 to 200 mA

Pause: Pacing pulse frequency reduced by a factor of 4 when activated

Refractory Period: 200 to 300 ms +/- 3% (function of rate)

ENVIRONMENTAL

Temperature, Operating: 5 to 45° C (41 to 113°F)

Temperature, Nonoperating: -20 to +60° C (-4 to +140° F) except therapy electrodes

Relative Humidity, Operating: 5 to 95%, noncondensing

Atmospheric Pressure, Operating: Ambient to 429 mmHg (0 to 4,572 meters) (0 to 15,000 feet)

Water Resistance, Operating (without accessories except for ECG Cable and hard paddles): IPX1 (spillage) per IEC 60601-1 clause 44.6

Vibration: MIL-STD-810E Method 514.4, Cat1

Shock (Drop): 1 drop on each side from 457.2 mm (18 in.) onto a steel surface

EMC

Emissions: MIL-STD-461D RE101; IEC 60601-1-2 (1993) subclause 36.201.1

Susceptibility: MIL-STD-461D CS114, RS101; IEC 60601-1-2 (1993) subclause 36.202

All specifications are at 20° C (68° F) unless otherwise stated.

► duration and voltage based on the patient's impedance level. In AED mode, the device can provide optional, escalating shocks (200–360 joules), depending on the patient's needs. In manual mode, the ALS clinician can give energy shocks according to established protocols.

QUIK-COMBO pacing/defibrillation/ECG electrodes are compatible with the entire LIFEPAK family of products, for standardization and continuity of care. Standard adult paddles with embedded pediatric electrodes, sterilizable adult paddles, and internal paddles provide flexible therapy options for response in various cardiac emergencies. The 20 is easy to maintain and service, comes with a five-year, in-hospital warranty, and is equipped with AC power and a backup internal nickel metal hydride (NiMH) battery.

The LIFEPAK 20 defibrillator/monitor extends and enhances the LIFEPAK family of products, providing flexible, compatible and standardized defibrillation solutions for patients across a range of hospital settings.



Medtronic Physio-Control
11811 Willows Road NE
P. O. Box 97006
Redmond, WA 98073-9706 USA
Tel: 425.867.4000
Fax: 425.867.4121
Internet: www.physiocontrol.com
Internet: www.medtronic.com

Europe
Tolochenaz, Switzerland
Tel: 41.21.803.8000
Fax: 41.21.803.8099

Switzerland
Tolochenaz, Switzerland
Tel: 41.21.803.8000
Fax: 41.21.803.8099

Canada
Mississauga, Ontario
Tel: 905.826.6020
Fax: 905.826.6620

United Kingdom, Ireland
Watford, Great Britain
Tel: 44.1923.212.213
Fax: 44.1923.241.004

France
Boulogne-Billancourt, France
Tel: 33.1.55.38.1700
Fax: 33.1.55.38.1800

Germany
Dusseldorf, Germany
Tel: 49.211.529.30
Fax: 49.211.529.31.00

Austria
Vienna, Austria
Tel: 43.1.240.44.160
Fax: 43.1.240.44.600

Italy
Milan, Italy
Tel: 39.02.66.16.41
Fax: 39.02.642.74.88

Netherlands
Kerkrade, The Netherlands
Tel: 31.45.566.8000
Fax: 31.45.566.8668

Spain
Madrid, Spain
Tel: 34.91.625.04.00
Fax: 34.91.650.74.10

Scandinavia
Järfälla, Sweden
Tel: 46.8.52.22.00.00
Fax: 46.8.52.22.00.50

Asia Pacific
Christchurch, New Zealand
Tel: 64.3.3794.429
Fax: 64.3.3792.374

Latin America
Sunrise, Florida USA
Tel: 954.835.4042
Fax: 425.885.6507

Middle East
Beirut, Lebanon
Tel: 961.1.370.670
Fax: 961.1.364.164

Hungary
Budapest, Hungary
Tel: 36.1.214.2228
Fax: 36.1.214.2230

Poland
Warsaw, Poland
Tel: 48.22.465.69.00
Fax: 48.22.465.69.17

Czech Republic
Prague, Czech Republic
Tel: 420.2.2017.2277
Fax: 420.2.2056.1617

People's Republic of China
Shanghai, China
Tel: 86.21.50800998
Fax: 86.21.50800978

South Africa
Bedfordview, South Africa
Tel: 27.11.677.4800
Fax: 27.11.616.1104

Japan
Kawasaki, Kanagawa, Japan
Tel: 81.44.540.6502
Fax: 81.44.540.6170

Australia
Sydney, Australia
Tel: 61.2.9879.5999
Fax: 61.2.9879.5100

